



DeRoyal®

Multidex® Gel/Powder Wound Care Pack

Single Patient Use Only
Contains Sterile and Non-Sterile Components
Rx Only

IMPORTANT

Read carefully and completely before using or applying.

GENERAL

Maltodextrin Wound Dressing is a sterile, Maltodextrin Powder Wound Dressing is a sterile, maltodextrin NF powder with 1% ascorbic acid. Multidex Powder mixes with wound exudate to form a gel that creates a moist environment conducive to healing. The gel creates an environment which helps the body's own cells to carry out the task of granulation tissue formation and the repair of the epithelium. Multidex eliminates odor from foul smelling wounds in 24 to 72 hours.

Multidex Gel Wound Dressing is a sterile, maltodextrin NF powder with 1% ascorbic acid, USP formulated into a gel using glycerin and water. Multidex Gel performs in the same way as the powder but has been formulated to help moisten or soften dry wounds.

INTENDED USE

Multidex Gel and Powder can be used on infected and non-infected wounds. They are indicated for all types of wounds, including dermal ulcers (e.g., leg ulcers, pressure ulcers and other secreting lesions), diabetic ulcers, abdominal wounds, infected wounds, superficial wounds, lacerations, cuts, abrasions, donor sites and second degree burns. Multidex Powder is indicated for moist wounds. Multidex Gel is indicated for moist to dry wounds.

INGREDIENTS

Multidex contains maltodextrin and 1% ascorbic acid. **NOTE:** No preservatives, dyes, additives, thickeners, fragrances or any other chemical ingredients have been added to Multidex.



DeRoyal®

DIRECTIONS

1. Prepare a clean work area in which to change your dressing.
2. Wash your hands with soap and water.
3. Put one pair of gloves on.
4. Remove the dressing that is currently on the wound. If dressing sticks to wound, soak with small amount of saline in pink bottle for several minutes before removing.
5. Rinse the wound using the small pink bottle of saline.
6. Gently dry the skin around the wound with the soft gauze pad.
7. Take old gloves off, put on clean gloves.
8. Fill the wound with Multidex.
9. Cover the wound with the dressing in the packet (Covaderm Plus® or Polyderm™ Plus).
10. Place all used items in the zip lock bag provided and place in regular trash.
11. Change the dressing daily or as instructed by your healthcare professional.

OBSERVATIONS

Multidex may cause a mild temporary burning sensation in some patients.



DeRoyal®

Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828

Part #4-2005
©2005 DeRoyal.
All Rights Reserved.
Revised 3/05